


Securement of Pala-Nate[®]

1. Wrap 1/4" wide medical tape around the endotracheal tube (ET) at a location to allow suturing to Elastoplast tape on neonate's upper lip.
2. Adhere 1/2" to 3/4" strip of Elastoplast to the neonate's upper lip.
3. Select the right or left side of the ET for placement of the first suture loop through the tape wrapped around the ET.
4. Suture from the tape through the bottom edge of the Elastoplast tape. Make sure the suture is tight.
5. Position the Pala-Nate in mouth as required, placing the Pala-Nate tab on top of the ET adjacent to the suture loop.
6. For best results, a second clinician should assist by holding the Pala-Nate tab securely on top of the ET. Place the second suture loop on the other side of the ET and through the Elastoplast while holding the Pala-Nate in place.
7. Making sure the tab of the Pala-Nate is between the sutures, wrap a second piece of medical tape over the tab and around the ET.
8. Procedure will take approximately three minutes.

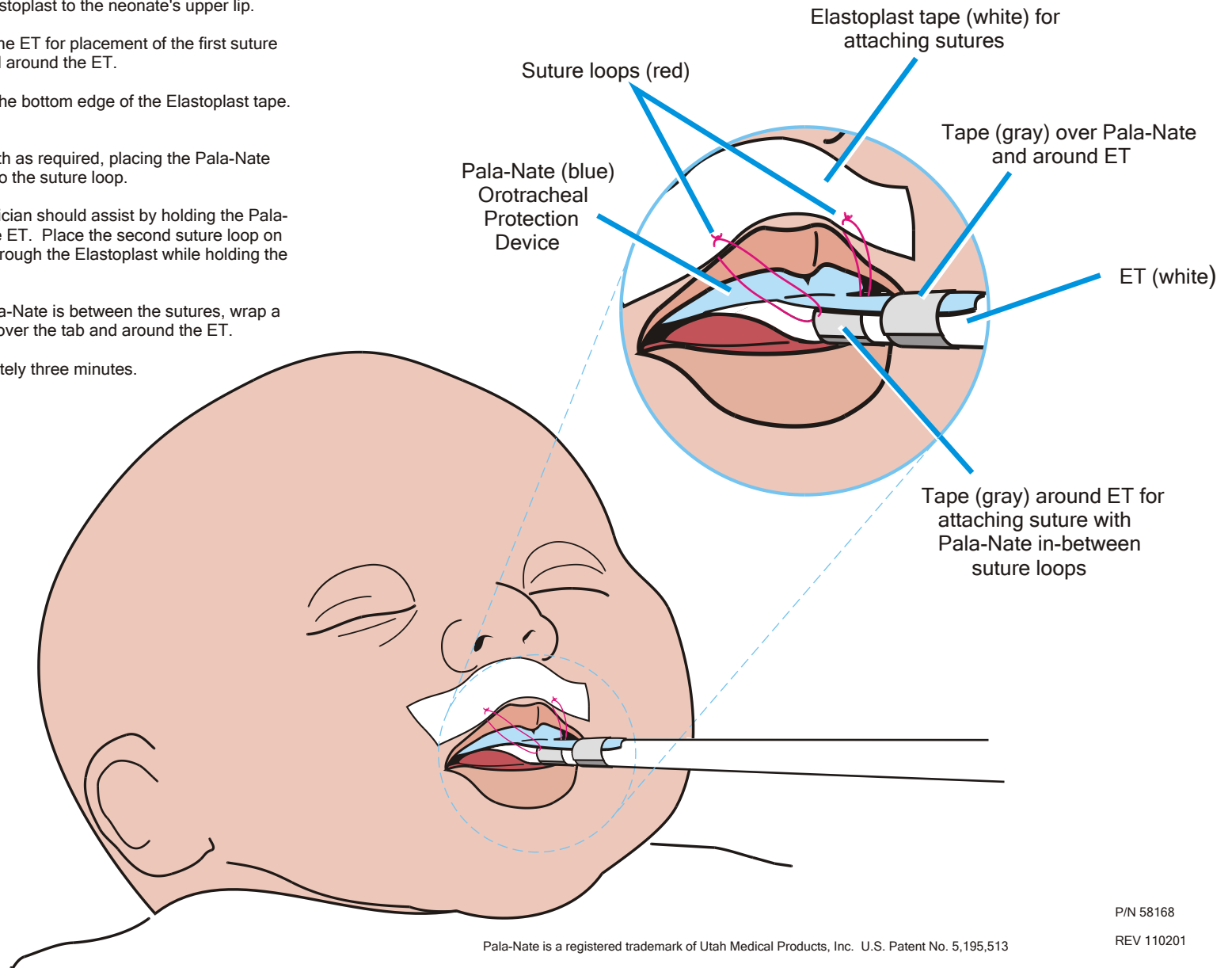

Utah Medical Products, Inc.
7043 South 300 West
Midvale, Utah 84047
Call Toll Free: 800 533-4984

Pala-Nate is a registered trademark of Utah Medical Products, Inc. U.S. Patent No. 5,195,513

P/N 58168

REV 110201