


SECURE HOLDING POWER AND CAPUT MOLDING

UTAH MEDICAL
PRODUCTS INC.




MUSHROOM-SHAPED CUPS

SEMI-RIGID POLYETHYLENE CUP WITH INVERTED LIP

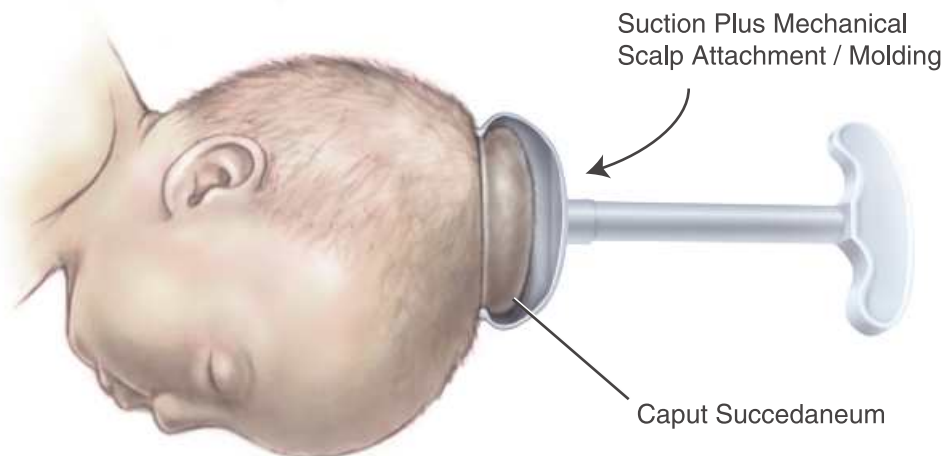

Contraindicated forced rotations on the fetal head are possible, as is traction in an off-axis direction, because the diameter of the caput succedaneum is greater than the inside diameter of the cup lip which creates a mechanical attachment to the head.

SOFT THERMOPLASTIC RUBBER CUP WITH STRAIGHT LIP

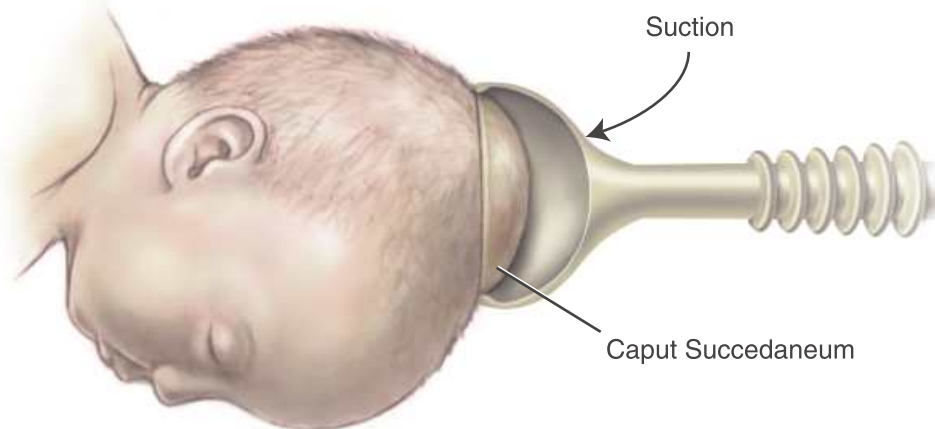

Contraindicated cup uses, such as forced rotation of the fetal head and traction in an off-axis direction, are not facilitated by the Secure Cup™, thus minimizing risks of scalp lacerations and other fetal injuries.